BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
GB: USER MANUAL
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
RO: INSTRUȚIUNI DE UTILIZARE
DE: GEBRAUCHSANWEISUNG
NL: GEBRUIKERSHANDLEIDING

HU: HASZNÁLATI ÚTMUTATÓ
ESP: INSTRUCCIONES DE USO
IT: ISTRUZIONI PER L'USO
РУ: ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
FR: NOTICE D'UTILISATION
SRP: UPUTSTVO ZA UPOTREBU

ELEGANCE DUO

XN-D212

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➤ **IMPORTANT!** READ CAREFULLY AND KEEP FOR FUTURE REFERENCE! THESE INSTRUCTIONS CONTAIN IMPORTANT INFORMATION, GUIDELINES AND RECOMMENDATIONS FOR THE PRODUCT AND ITS SAFE USAGE.

Congratulations on the purchase of this electric pump. This product is an electric breast pump with different levels of control of the suction power, unique valve with gentle suction and soft massage silicone gel pad. The Electric breast pump complies with EN55014-1 and EN61000-3-2;3-3.

The Cangaroo electric breast pump provides many of the features which the busy mothers love and is perfect for many reasons:

- Expressing and collecting breast milk while you are away from your baby, for example when you are at work.
- Relieving the tension in your breasts caused by the large amount of breast milk. For example, when your baby's feeding habits change because it has started eating more solids.
- Maintaining your milk supply if you have to temporarily stop breastfeeding. For example, to express the unsuitable milk while you are undergoing medical treatment.

The Cangaroo electric breast pump has the following features:

I. Massage cycle, so called stimulation - 10 Levels and 3 frequency Levels

This way the user can adjust the most suitable frequency of expression and stimulation, because the range of the power of expression varies from 7 to 16 kPa, with frequency from 70 to 90 times/minute.

II. Expression cycle - 15 Levels with 3 levels of frequency. The user can adjust to the most suitable power and frequency of expression, because the power of expression varies from 10 to 45 kPa.

III. Quiet motor for discretion.

IV. Light and easy to carry due to its small size.

V. Easy to clean

I. RECOMMENDATIONS AND WARNINGS FOR THE SAFE USAGE OF THE PRODUCT

1. The appliance is not intended for use by children. Keep the appliance and its adaptor out of reach of children.

2. Children shall not play with the appliance.

This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and if they understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children.

4. WARNING: To reduce the risk of electrocution, please:

- Do not use while bathing or showering.
- Do not place or store this product where it can fall or be pushed into a bathtub or sink.
- Do not place or drop into water or other liquid.
- Do not reach after a product that has fallen into water. Unplug from the wall socket immediately.

5. WARNING: To reduce the risk of burns, electrocution, fire, or injury:

- Close supervision is necessary when the product is used near children and disabled persons.
- Use this product only for its intended purpose as described in this manual. Do not use attachments, which are not recommended by the manufacturer and do not make any improvements or changes in the product as this may lead to its malfunction or injuries to those who use it.
- Never operate this product if it has a damaged adapter or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
- Keep the adaptor away from heat sources.
- Never use just before falling asleep or if you are feeling drowsy.
- Be careful not to drop the product and do not insert any other objects into the opening or tubing.
- Make sure the voltage of the AC adaptor is compatible with the power source.
- Only use the AC adaptor which is included in the set. If any damage of the adaptor occurs, please, contact the manufacturer or the sales agent, from whom you have purchased the product.

If you are infected with Hepatitis A, Hepatitis B or you are HIV positive (Human Immunodeficiency Virus),

pumping breast milk does not reduce or remove the risk of transmitting the virus to your baby through breast milk.

- Do not operate a motor vehicle while using the electric breast pump.
- Do not attempt to modify or repair the product by yourself when damage occurs. Instead you must contact an authorized service or the sales agent, from whom you have purchased the electric breast pump.
- This appliance is intended for repeated use only by one user.
- Do not allow children or pets to play with the product, its adaptor or the accessories included in the set.
- Before each use inspect the pump and the adaptor for signs of damage. Do not use them if you find any damage.
- Do not use spare parts which are not the original ones supplied by the manufacturer or importer.
- This product is a personal care item and should not be shared between mothers.
- Do not attempt to remove the pump body from your breast while under vacuum. Turn the unit off and break the seal between your breast and the silicone pad with your finger, then remove the pump from your breast.
- Never use while pregnant as pumping can induce labor.
- Do not leave and do not store the product in a place with very high room temperature or under direct sunlight.
- The product should not be left unattended when plugged into the electrical wall socket.
- Before starting the breast pump, first connect the adapter to the electric breast pump, then securely connect the adaptor to the electric socket.
- Wash, rinse and sterilize the appropriate pump parts before each use.
- Never wash and sterilize the electrical parts of the product – the motor of the pump and the AC adaptor in order not to cause permanent damage to the product.

II. PARTS - PD

1.Safety Cover; 2.Silicone breast pad; 3.Cap; 4.Piston; 5.Connecting tube; 6.Pump body; 7.Air hole; 8.Valve; 9.Pumping mechanism (motor); 10.Feeding bottle; 11.Charger/ adapter; 12. USB cable; I and J. Connecting elements;

III. DISPLAY - D

A. Massage mode; B. Suction mode; C. Time; D. Frequency; E. Levels; F. Symbol of mode

IV. PANEL - P

A. Level decreasing; B. Mode switch; C. Turning on/off; D. Level increasing; E. Frequency;

V. ELECTRICAL CHARACTERISTICS

- Battery voltage and capacity:** 5 VDC/ 2000 mAh
- BATTERY CHARGER: Input:** AC115V – 230V; **Output:** 5VDC, 1.0 A
- Charging time:** 100 minutes
- Method of control:** Touchscreen
- Display:** LCD

VI. CHARGING THE ELECTRIC MOTOR OF THE PUMP

- 1.Before first time use charge the battery completely.
- 3.The breast pump is with a built-in battery 5V/ 2000 mAh.

3.Charging:

-Connect the adapter to the charging jack of the pumping mechanism.

-Insert the plug into electrical outlet of the mains supply of your home.

-We recommend you to leave the breast pump to charge completely before next use in order to prolong its battery life.

4. Use only the charger provided with the breast pump. Do not use chargers from other products.

5. After the battery is fully charged the indicator will stop flashing.

6. If you will not use the product for a long time, charge the battery completely before storing it. Then charge the battery at least once a month to prolong its life.

7. If the battery is damaged or cannot be charged anymore, do not try to open the product by yourselves. For consultation and repair contact the shop from where you bought the product or the importer/ distributor.

8. The battery is recyclable. If the product is damaged and cannot be used any longer, take it to an official collection point for electronic products' waste, batteries, chargers. DO NOT throw the product away with the household waste. Follow your country's rules for the separate collection of electrical and electronic products and rechargeable batteries. Dispose of waste of this type only at the designated places. This will be important and significant contribution for the environmental protection and the reduction of the harmful effects of the substances contained in batteries and electronic components on the environment and human health.

VII. ASSEMBLY OF THE ELECTRIC BREAST PUMP

Follow the figures shown for assembly of the product in order to ensure its correct assembly:

Step 1: Put the silicone valve (fig. 8) in the lower end of the pump body. Make sure it is fixed properly around the edges of the pump body opening, intended for it. **Warning! If the valve is loose or not properly installed, not enough vacuum can be created!**

2. Step 2: For better fix of the pump, put the feeding bottle (10) in its base (6). Turn it gently in clockwise direction so that the groove of the bottle fits exactly the groove of the body of the pump.

3. Step 3: Carefully put the piston (4) in the pump body. Make sure the fit is tight around the edges of the opening of the pump's body, intended for it.

4. Step 4: Put the protective cover at the upper part of the pump body. Press and turn in clockwise direction so it fits in the correct position.

5. Step 5: With clean hands put the silicon piston (2) in the pump body (6). Make sure that it fits tightly. Advice: If you are not going to use the breast pump, place the dust cover (1).

6. Step 6: Put one end of the tubing(5) in the pump body(6). After that put the other end of the tubing in the pumping motor of the pump (9). Now you can use the breast pump.

➤ **WARNING! DO NOT STERILIZE AND DIP INTO WATER THE PUMPING MECHANISM AND THE AIR TUBE, THAT CONNECTS THE MECHANISM TO THE PUMP' BODY – PART 5.**

VIII. USAGE INSTRUCTIONS

1. Wash your hands thoroughly and make sure your breasts are clean. Squeeze a little breast milk from each breast to ensure there's no blockage. Make sure your breast pump is sterilized and properly installed.

2. Relax in a comfortable chair, leaning slightly forward (use cushions to support your back).

3. Place the pump on your breast and turn the pumping motor on.

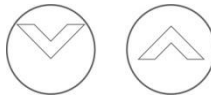
A. Massage (stimulation):

This function helps the stimulation of the milk flow.

We advise to use this option first use this option and then switch to expression mode.


The massage mode imitates baby suckling, not too strong and stimulates milk flow. The expression mode also imitates baby suckling, but stronger than the first mode, more even and with great efficacy.

Press the button for selection of mode and activate the massage function. With "+" and "-" buttons you can adjust the stimulation levels. They are 10 in total. With the frequency button – the frequency levels. They are 3 in total.

B. Expression:

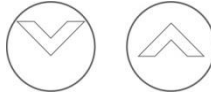Press the button for selection of mode and activate the expression function. With “+” and “-” buttons you can adjust the levels of expression. They are 15 in total. With the frequency button – the frequency levels. They are 3 in total.

C. Memory

Plug the adapter into the pump after you stop using it and the frequency and level of suction you have used last will be memorized. When you turn the breast pump on, it will start working at the same frequency and level.

- **If there is no result from the pumping, please stop trying! Try to use the pump at another time of the day.**
- You should pump for 8 minutes on average, for 120 ml to get expressed. Please, have in mind that the quantity of milk varies for different women. If you find that you usually express more than 120 ml. per session, you should use larger feeding bottle.
- **Do not overfill the feeding bottle, otherwise it may leak.**
- When you stop using the pump, switch off from the ON/ OFF button. Afterwards remove the tube from the rear part of the pump's body. Remove the body of the pump from the bottle. Close the bottle with the spiral lid and the nipple with cover.

IX. WHEN TO USE YOUR BREAST PUMP

1. If it is possible. Only if you have sufficient milk secretion and normal time for breast feeding. The use of the electric breast pump may begin usually two or four weeks after the birth of your baby.
2. In the following cases you can always use your breast pump:
 - When breast milk must be provided to your baby, but you are in the hospital.
 - If you have more breast milk, you can express some of it between meals in order to reduce the pain of the bulging breasts, and this way it is easier for the baby to suck.
 - If your nipples pain or are cracking, you can express some milk for feeding until the nipples get back.
 - If you need to separate with your baby and hope to continue breastfeeding after reunion, breast milk can be expressed periodically to maintain milk secretion.

X. HINTS

It takes practice in order to use the breast pump the best possible way. So it will take you at least several attempts before starting successfully to use the electric breast pump.

1. Choose a quiet time to use the pump – time, when no one will disturb you.
2. Sit in a comfortable chair, in a perpendicular position to the ground. Please, put a comfortable cushion behind your back before you start using the breast pump. Take a glass of water and place it near you, because you may feel thirsty during the pumping process.
3. Place a warm cloth on your breasts for a few minutes before starting to express the milk as this may facilitate the process and relieve the tension in the breasts.
4. Warmth and relaxation can help for the easier pumping of the breast milk, so before using the pump, you might like to take a bath or a shower.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after the feed.

6. Repositioning the pump on your breast from time to time can help stimulate the secretion. Practice in order to find the best pumping technique for you. If the process is causing you breast pain, stop immediately and consult with a doctor at once.

7. Make sure that you have not blocked or covered the air opening of the pump body.

8. Breast pump use errors:

The electric pump is an apparatus, which helps postpartum women to ease the pain and to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. For some women, which are mothers for the first time, the mammary gland usually is not smooth enough. A reason sometimes can be the congenital nipple inversion or milk water shortage. So, we suggest you before purchasing and using the pump, to consult with a doctor who could confirm that your mammary gland is smooth and that the breast milk is in normal condition. Do not buy and use the pump only as a device to unblock your milk ducts.

XI. HOW TO STORE THE BREAST MILK

1. Only milk collected with a sterile pump should be stored for feeding Your baby.

2. Breast milk can be stored in a bottle warmer at 40°C, so you can breast feed Your baby within 4 hours.

3. If you are storing freshly expressed milk, please, seal it, the breast milk can be stored for 10 hours in room temperature 19 - 25°C.

4. Expressed breast milk can be stored in the refrigerator for up to 48 hours (but not in the door) or in the freezer for up to 3 months.

5. Frozen breast milk must firstly thaw in the refrigerator naturally. If you need it urgently, put it in a kettle and thaw it. After thawing, the breast milk can be stored in a refrigerator for 24 hours.

6. NEVER REFREEZE THE BREAST MILK.

7. The frozen breast milk can be heated gradually in warm water or in baby bottle warmer.

8. NEVER HEAT THE MILK IN A MICROWAVE AS THIS MAY DESTROY VALUABLE NUTRIENTS AND ANTIBODIES.

In addition, the uneven heating may cause hot spots in the liquid which could cause burns to your baby's throat. Never immerse a frozen milk container in boiling water as this may cause it to crack/ split. Never put boiling water directly into a feeding bottle, allow to cool for around 20 minutes before filling.

XII. CLEANING AND STERILIZATION OF THE PUMP

➤ **WARNING! DO NOT** STERILIZE OR DIP INTO WATER THE PUMPING MECHANISM. CLEAN IT ONLY WITH DRY SOFT CLOTH.

➤ **Clean and sterilize** the parts of the pump coming in contact with your breasts and the breast milk before initial use and after each use, follow the instructions below.

➤ **DO NOT** WET OR STERILIZE PART 5 – THE TUBE CONNECTING THE PUMP BODY TO THE PUMPING MECHANISM.

➤ **Sterilization method:**

1. Pull the tube (5) from the Pumping body (9) and relocate it elsewhere. Separate all parts, 1, 2, 3, 4, 6, 7, 8, 10 then wash in warm soapy water and rinse thoroughly. Do not use antibacterial or abrasive cleaners/detergents.

2. Sterilize ONLY the following parts 1, 2, 3, 4, 6, 7, 8, 10 in steam sterilizer or by boiling for 10 minutes.

❖ CLEANING SUGGESTIONS

1. Use steam sterilizer for sterilizing the pump. Boiling is not recommended. The first reason is that the product is easy to damage, if exposed to improper sterilization. Second, this unscientific method of sterilization cannot achieve the complete effect.

2. If you decide to use boiling as a method of sterilization, please, pour more water into the pot so that the parts can be completely covered with water. Do not leave the product unattended during sterilization. Do not boil for too long – maximum 10 minutes.

3. For the nursing mothers who breastfeed once every 2 – 3 hours, it is not recommended to sterilize and clean after each use. Excessive disinfection and cleansing can easily lead to premature aging and reduce of the life of breast pump. It is recommended to thoroughly clean the stains and sterilize once a day.

4. Store the parts in a clean container.

➤ **IMPORTANT!** If the breast pump is for the purpose of storage of the milk, you should take care of the disinfection of all washable parts of the pump, otherwise the milk may go bad, as it is a product difficult to store.

➤ **WARNING!** Before use you should make sure that the body of the breast pump, the piston, the valve and the cover are dry.

➤ **We recommend you not to sterilize the tube – part 5 in order to avoid accidental entry of water into the pumping motor.**

➤ If you use a steam sterilizer for cleaning of the parts of the pump, you should be careful not to enter in contact with the heater of the sterilizer. If you use boiling as a method for sterilization, be careful not to enter into contact with the bottom of the vessel in which you boil.

XIII. TROUBLESHOOTING GUIDE

1. Lack of suction:

- Check if you have assembled all parts correctly and if the valve is positioned correctly.
- Ensure that the silicone massage pad and the shield brace are fitted securely to the pump body and a perfect seal is created.
- Check if you have placed the silicone valve firmly to the opening of the pump body.
- Check whether there are any damaged parts.
- If you cannot turn the pump on, check whether it is connected to the adapter and whether it is plugged in the electricity grid.

2. No milk is being expressed:

- Check point 1
- Ensure the pump is correctly assembled and that suction is being created.
- Relax and try again.
- There are 15 levels of expression, the maximum power is 42KPa; the average suction power of the baby is 20 – 30 kPa. We recommend you increase the levels gradually; the most suitable level is the one, with which the suction is painless. **The following is NOT true: the stronger – the better.** Too strong suction can hurt the nipple.

3. Pain in the breast area where there is expression:

- If you feel pain and no milk flows during expression, maybe your mammary gland is blocked. This case may result in mastitis. Contact a physician promptly. The complete milk expression will have good effect on mastitis protection.
- You may be using too high level of suction. You do not need to use all the suction that the pump can generate.
- Try to adjust the proper suction and frequency.
- Consult your health professional/ breastfeeding advisor.

4. Flow is too slow:

The speed of milk flow differs for every woman. The higher degree and level of expression do not mean faster flow of milk. This depends on many factors – constitution of the breast, level of pass ability, even the mood. The more regularly you use the breast pump, the quicker and easier the flow of milk. Before you start using it, put a warm cloth on your breasts and use the massage mode for about 5 minutes before you switch to expression mode.

5. Milk is dropping down the silicone massage pad:

- Remove and refit again the massage pad in order to ensure it is firmly attached to the central part of the funnel.
- Try to lean slightly forward.

6. Cracking and discoloration of the pump/ feeding bottle parts:

Avoid contact with antibacterial or abrasive detergents/ cleaners as they can damage the plastic. Combination of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations can cause plastic to crack. If this occurs, please, do not use. The pump body and the breast milk containers are dishwasher safe but the food colorings may cause discoloring them. Keep the product out of direct sunlight as

prolonged exposure
may lead to discoloration.

7. Lost or broken parts:

For spare parts contact an authorized service center or the sales agent from whom you have purchased the product. Do not use spare parts, which are not approved or supplied by the manufacturer.

•Do not attempt to repair or to modify the product, because this may endanger Your and Your baby's health.

Thank you for purchasing this product! Use it carefully and follow the instructions from this manual in order to ensure the safe use and maintenance of the electric breast pump. When having questions regarding the use, maintenance and repair of the product, please, contact the importer or sales agent from whom you have purchased the product.

Contents: PP and silicone PRODUCED FOR CANGAROO

Manufacturer and Importer: Moni Trade LTD

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