
BG: ИНСТРУКЦИЯ ЗА УПОТРЕБА
EN: USER MANUAL
DE: GEBRAUCHSANWEISUNG
GR: ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
RO: INSTRUCTIUNI DE UTILIZARE
ES: INSTRUCCIONES DE USO

IT: ISTRUZIONI PER L'USO
FR: NOTICE D'UTILISATION
NL: GEBRUIKERSHANDLEIDING
РУ: ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
SR/ MNE: UPUTSTVO ZA UPOTREBU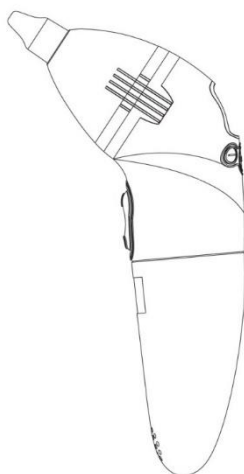

Purity

NC001


BG: ЕЛЕКТРИЧЕСКИ АСПИРАТОР ЗА НОС
EN: ELECTRIC NASAL ASPIRATOR
DE: ELEKTRISCHEN NASENSAUGER
GR: ΧΕΙΡΙΣΜΟΣ ΡΙΝΙΚΟΥ ΑΝΑΡΡΟΦΗΤΗΡΑ
RO: ASPIRATOR NAZAL ELECTRIC
ES: ASPIRADOR DE NARIZ ELÉCTRICO
IT: ASPIRATORE NASALE ELETTRICO
FR: MOUCHE-BÉBÉ ÉLECTRIQUE
NL: ELEKTRISCHE NEUSREINIGER
РУ: ЭЛЕКТРИЧЕСКИЙ НАЗАЛЬНЫЙ АСПИРАТОР
SR/ MNE: ELEKTRIČKOG ASPIRATORA ZA NOS


0123

Описание на частите на уреда/ Parts description/ Gerätebeschreibung/ Περιγραφή της συσκευής/
 Descrierea aparatului/ Componentes del product/ Descrizione delle parti dell'apparecchio/
 Description des pièces de l'appareil/Omschrijving van de onderdelen van het toestel/ Описание
 частей прибора/Opis komponentata uređaja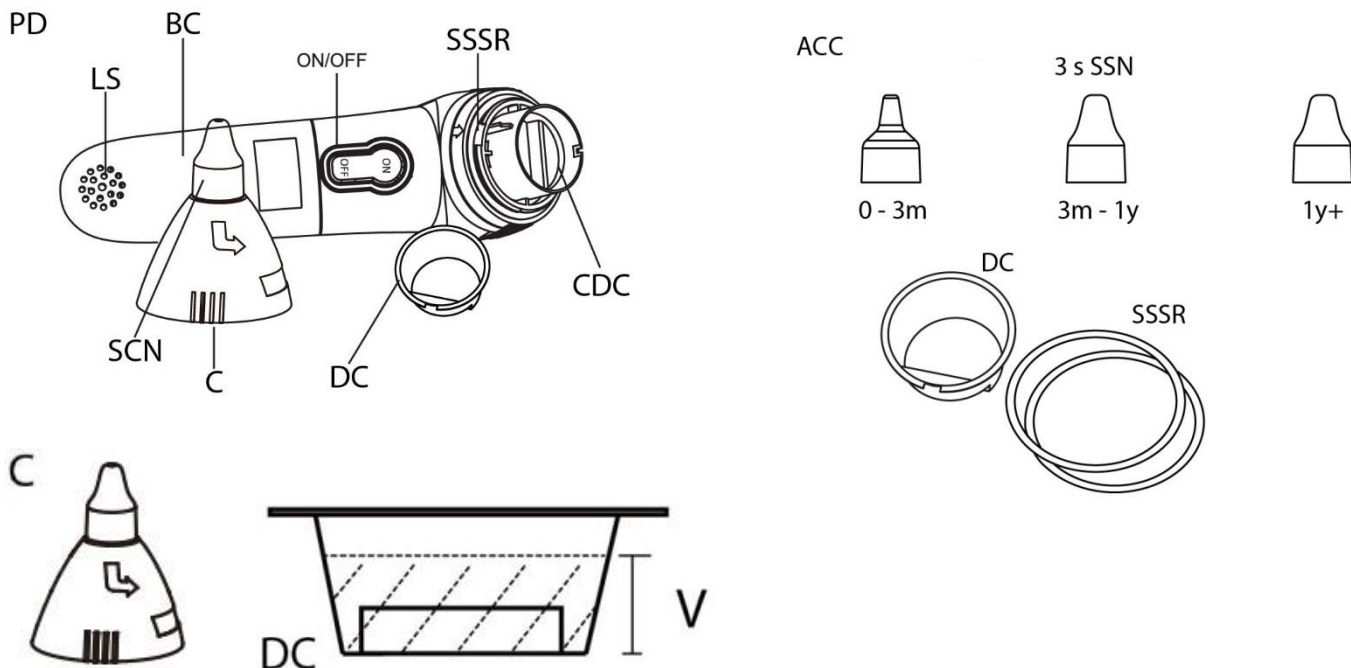

Настройки и функции/ Settings and functions/ Menüeinrichtung und Funktion/ Ρύθμιση και
 λειτουργία του μενού/ Setarea și funcția meniului/ El uso/ Impostazioni e funzioni/ Réglages et
 fonctions/ Instellingen en functies van het menu/ Настройки и функции/ Podešavanje i funkcije

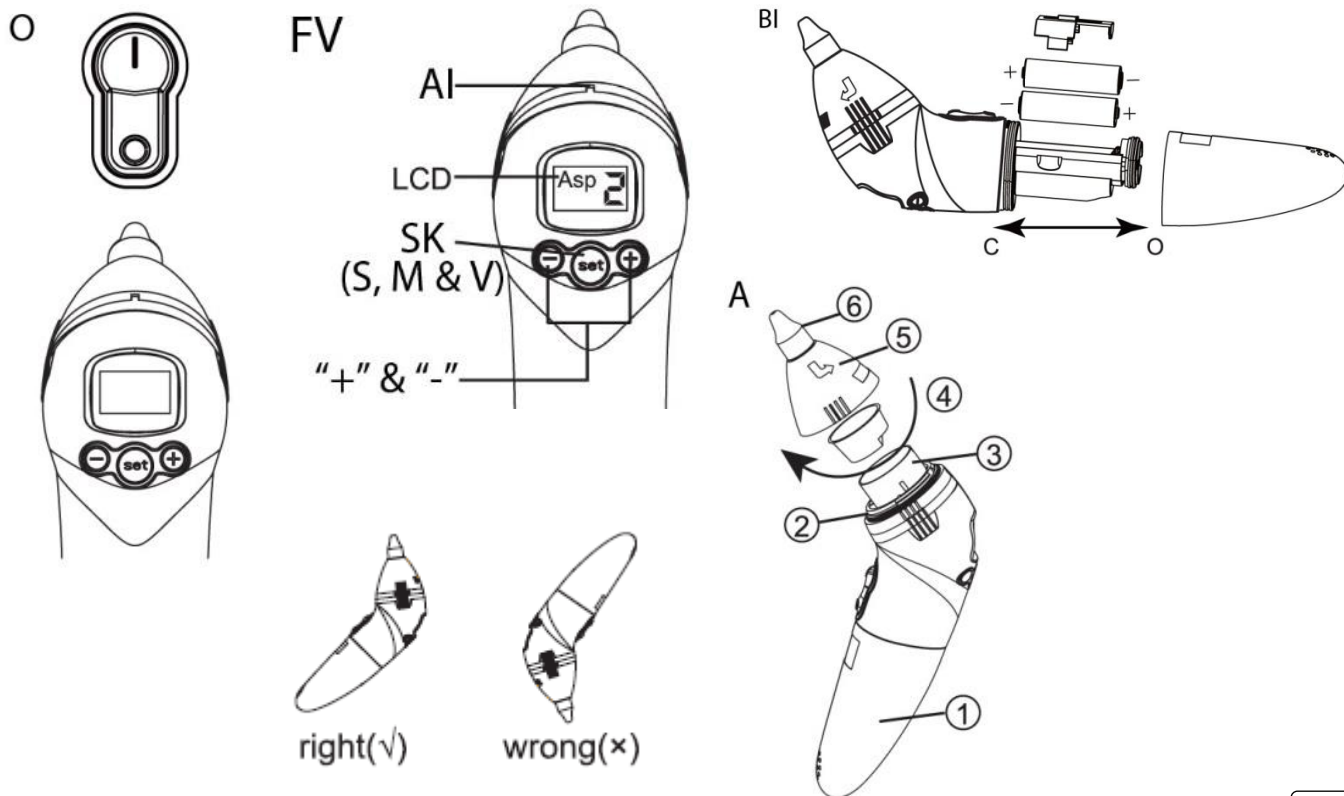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Адрес: 38 Huanzhen Xi Road Dagang Town, Nansha, 511470 Guangzhou, Guangdong, PEOPLE'S REPUBLIC OF CHINA

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Адрес: Eiffestrasse 80, 20537 Hamburg, Germany

➤ **THE MANUFACTURER KEEPS THE RIGHT TO CHANGE SPECIFICATIONS OF THE PRODUCT WITHOUT PRE-WARNING!**

➤ **IMPORTANT! READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!**

The nasal aspirator is a baby care product, which possesses lots of advantages, like high efficiency and easy operation. Since the babies no more than one year old can only breathe through the nose, it is essential to clean their nasal secretion and mucus regularly in case of illness.

I. RULES AND RECOMMENDATIONS FOR SAFETY USAGE

1. Only use nasal aspirator once you have read and understood the instruction for use.
2. Only use nasal aspirator for the specific purpose described in the instruction manual. Any misuse will void the warranty.
3. Nasal aspirator always be used for personal home use.
4. Using nasal aspirator is no substitute for consulting a doctor. Do not self-diagnose or self-medicate without consulting your physician. In particular, do not start taking any new medication or change the type and/ or dosage of any existing medication without prior approval.
5. In the case of infection, fever, nasal irritation, or hemorrhage, stop using nasal aspirator and consult your physician.
6. Nasal aspirator must be operated by adults. Keep it out of the reach of children.
7. Do not allow children to play with the device. It is not a toy. Risk of choking! Contains small parts. Danger of swallowing or choking.
8. Before and after each use the nasal aspirator should be cleaned and disinfected according to the requirements of this manual to avoid possible bacterial infections.
9. The device is intended to clean the nose. Do not put it in the ears, or eyes or the mouth of the child or other places on its body.
10. When using this device, there will be some noise and vibration caused by the device. This is normal and does not indicate a malfunction.
11. Do not insert the silicone nozzle too far into the nostrils.
12. Do not use for a long time. Few seconds of cleaning the nasal secretion is enough.
13. Use only the silicone suction nozzles provided by the manufacturer.
14. Do not expose the nasal aspirator to electric shocks.
15. Do not store the nasal aspirator in extreme temperature conditions of greater than 55°C (131°F) or less than -20°C (-4°F).
16. Do not store the nasal aspirator in relative humidity higher than 95%.
17. Do not put the nasal aspirator in a place with direct sunlight, high temperature, high humidity, excessive dust, or place where it may contact, corrosive gas or liquid, and do not use this product in the above environment.
18. Do not drop the nasal aspirator.
19. Do not use the nasal aspirator if it is damaged.
20. Do not open up the nasal aspirator. It contains small parts, which can be swallowed by children.
21. Never immerse the nasal aspirator in water or other cleaning liquids, and do not let any liquid to enter the interior of this product.
22. **WARNING:** Modification of this device is not allowed.
23. If you will not use the product for a longer period than 14 days, take out the batteries in order to avoid their leakage. Follow the instructions for the battery change in this manual.
24. Keep the nasal aspirator away from electromagnetic fields produced by objects such as microwave ovens, arc welders or induction heaters.
25. When you use the appliance, make sure your child is with opened mouth in order to keep the air ventilation inside its nose in case of negative air pressure inside the baby's head and unease.
26. The device is not with function for backflow, so always use the disposable cups. This way you will prevent the access of mucus inside the device and its damage.
27. The cups for the mucus are single use only. Change the cup when the mucus reaches two thirds of the volume of the cup (See figure DC).
28. When the device is working or if there is already mucus in the cup, do not turn the aspirator upside down. Otherwise, the mucus can enter the motor of the aspirator and damage it (See Figure right/ wrong).
29. Clean the nozzle well (Figure C) or leave it to dry completely after sterilization and before use. If it is not completely dry, drops of water may enter the motor and damage the device.

30. When you attach the silicone ring back, you should clean the place, where the ring will be attached, well from dust. Make sure the ring fits completely. Otherwise, the suction power will be adversely affected.
31. The nasal aspirator should no longer be reused when it reaches its expected service life (5 years, from the date of production) of the device. See chapter XVI to dispose the device.
32. Electrical components and batteries contain heavy metals and chemicals, which may do harm to the environment. Follow the local law and regulation on the disposal of the device.
33. Contact the importer or the shop from which you bought the product if the following problems occur:
 - If an error occurs or if nasal aspirator is damaged. DO NOT attempt to repair nasal aspirator yourself, as this will void the warranty. Only have repairs carried out by authorized service partners. The manufacturer will provide parts and instructions to the authorized service partners needed to repair the device.
 - If you need assistance in setting up, using, or maintaining the nasal aspirator.
 - If you have unexpected operation or events to report.

II. INTENDED PURPOSE

- **Intended Use:** The device is intended to remove excess mucus from the nose of children.
- **Intended for:** Children with colds or rhinitis (including infants).
- **Intended users:** Adult (above 18 years old).
- **Use environment:**
 1. Normal using condition
 - Ambient temperature: 10°C ~ 40°C (50°F ~ 104°F);
 - Relative humidity: ≤ 85%RH;
 - Pressure altitude: 700hPa ~ 1060hPa;
 2. Storage and transport conditions
 - Ambient temperature: -20°C ~ 55°C (-4°F ~ 131°F);
 - Relative humidity: ≤ 95%RH;
 - Pressure altitude: 700hPa ~ 1060hPa;

III. CONTRADICTIONS

Do not use nasal aspirator when the patient has the following symptoms:

- 1.) Nose bleed.
- 2.) Acute head, face or neck injury.
- 3.) Coagulation disease or hemorrhagic disease.

IV. RECOMMENDED OPERATING TIME

Note that the product's operation mode is intermittent. It is recommended to use the product from 5 to 10 minutes.

V. DEVICE STRUCTURE AND ACCESSORIES

- Check PD: BC – Battery compartment cover; LS – speaker; ON/ OFF – Button for ON/OFF; SSSR – Silicone sealing ring (1 pcs + 2 pcs spare ones); CDC – secretion cup compartment; DC – disposable cup (25 pcs included); C – cover; SCN/ 3s SSN – silicone nose probe (3 pcs with different sizes);
- Check Figure FV: AI – Air vent; LCD – Display; SK (S,M, & V) – button for selecting functions (suction, melody & volume); „+“ & „-“ – adjusting suction/volume;

VI. TECHNICAL SPECIFICATIONS

1. Recovery Time: 2 hours (The time required for the nasal aspirator to adjust to the temperature of the room in which it will be used, after taking it out of its storage place, when it was not used and where the temperature varied in the diapason of: -20°C ~ 55°C.)
2. Batteries: DC 3V (2 pcs AA alkaline batteries)
3. Product size: 196 x 70 x 53 mm (L x W x H)
4. Product weight: 185 grams (without batteries)
5. Maximum suction: 58 kPa.
6. Sound: noise level: ≤80 dBA (patient at 0.1m distance; operator at 0.3m distance).
7. Longevity of the Product: for approx. 20 000 takings (around 5 years from the date of manufacturing it).
8. Applied part: silicone suction nozzle.
9. Essential performance: suction.

VII. BATTERIES

- 1.Battery change:** When on the LCD display this symbol „  “ shows, it means the batteries are exhausted and should be changed with new ones. Use 2 new batteries size AA. See figure BI: C – close; O – open
- Open the battery cover by pushing it in the shown direction in figure BI.
 - Insert 2 AA batteries in the compartment correctly according to the polarity indicated on the bottom of the battery compartment. If you do not place the batteries correctly, this may lead to damaging the device. Do not use rechargeable batteries.
 - Place back the battery cover on the battery compartment.
- 2.DISPOSING OF BATTERIES:** To protect the environment, dispose of empty batteries at appropriate collection sites according to national or local regulations.
- 3.IMPORTANT GUIDELINES FOR BATTERIES:**
- The battery must be handled by an adult
 - Batteries are not a toy. Do not allow children to play with them.
 - Keep batteries out of the reachable zone of children! Risk of ingestion and strangulation!
 - Always remove exhausted batteries and do not dispose of them with household waste, but in designated areas.
 - Use only AA size batteries.
 - Alkaline batteries are recommended.
 - When replacing with new batteries, always replace all batteries. Do not mix old batteries with new batteries..
 - Children should not be present while changing batteries.
 - Always insert the batteries correctly according to the polarity (+/-) indicated on the bottom of the compartment. Failure to put them correctly can result in a short circuit or electric shock...
 - Do not mix alkaline batteries with standard (Carbon-Zinc) or rechargeable (Nickel-Cadmium) batteries...
 - Do not dispose of batteries in a fire as they may explode or leak
 - Keep batteries away from direct heat sources - direct sunlight, heaters or cookware or open flames. Do not dispose of batteries in a fire as they may explode or leak.
 - If you are not going to use the appliance for a long time, remove the batteries. This will protect the device from battery leakage and corrosion. .
 - If the batteries are found to be leaking when replaced, wear protective gloves and clean the battery compartment with a dry cloth. .
 - Non-rechargeable batteries should not be charged..
 - Do not disassemble, open or fragment the batteries.

VIII. ASSEMBLE THE DEVICE

See figure A:

- After inserting the batteries (1), place the secretion cup (4) on the base (3).
- Make sure the silicone sealing ring is properly fitted (2), allowing a good vacuum to be created.
- Place the cap (5) over the secretion cup and rotate it to secure it.
- Put on the top of the lid a silicone suction nozzle:
 - Pink is intended for newborns up to 3 months age.
 - The blue nozzle is for a baby from 3 to 12 months of age.
 - Orange is for children over 12 months of age.

IX. INSTRUCTIONS FOR USE

➤ **Understand the features of the product well before you start using it. Check figure O:**

- Place the batteries.
- Make sure that the silicone suction nozzle is attached correctly and fits tightly to the rib of the cover

Warning: After each use check the silicone nozzle. If you see any signs of tear or damage, change the nozzle immediately.

The silicone suction nozzles belong to consumables, their service life depends on the frequency of use of the device. We recommend replacing silicone suction nozzle every six months of use.

3.Turn on the device by pressing the ON button. Press the SET button to choose the functions and hold it for 3 seconds to enter ASP mode. The ASP letters will show on the screen. Press the “+” or “-” buttons to choose the aspiration strength (3 levels: 1 – gentle; 2 – medium; 3 – high)

4.Place and hold the child in upright position or in a position in which it is leaning slightly back. The child should not be in a laying position. Carefully put the silicone nozzle into one of the nostrils of the child. Be careful not to press the inner lining of the nostril or to place the tip of the nozzle too far into the nostril. The nozzle should not come into contact with the nasal membrane.

5.During use hold the nasal aspirator at an angle of 40° – 50° of the child’s face. The mucus must be cleaned by the aspirator without any additional external force.

Warning: Do not suction more than 6 seconds. Instead perform several short suction each for 3 to 5 seconds.

6. When you finish cleaning the mucus, turn off the device by pressing the OFF button.

7. Clean and the other nostril if needed. If the child feels discomfort or pain, stop using the aspirator immediately.

8. Clean the device after each use (See XI. Cleaning and Disinfection).

9. During use you can activate the music mode by: After choosing the ASP mode, Press again the SET button, to enter MUS mode. Press the “+” and “-” buttons to choose the melody you like (0 – no melodies; 1/2/3 – option 1/2/3). Press the SET button to enter VOL model. Press the “+” and “-” buttons to choose the desired sound volume. (VOL 1, VOL 2, VOL 3).

Menu		0	1	2	3
ASP	Aspiration	-	30% Gentle	70% Average	100% High
MUS	Music/Melody	Off	Option 1	Option 2	Option 3
VOL	Volume	-	Low	Average	High

10. To restart or return the preset, choose ASP: 2/ MUS: 1/ VOL: 2.

When the nasal aspirator is turned on, it will automatically start with the default state.

Warning: Lift the child’s head to ensure easier air flow.

Warning: Moisture the nasal secretion before using the aspirator.

Warning: Make sure the child’s mouth is open during use of the aspirator so that you ensure easier air flow.

Warning: After each use throw away the disposable cup. This cup is intended for single use only.

X. TROUBLESHOOTING

If any of the below listed issues occurs, check the possible solutions listed in the table. If you do not manage to resolve the issue, contact the shop from which you bought the product, for help. Do not try to repair the product by yourself, because the warranty will be void!

ISSUE	SOLUTION
The device does not do any suction	1. Check if the sealing ring is correctly attached. 2. Check if the transparent cover is correctly attached or not. 3. Check if the silicone nozzle is properly fitted or not.
When you press the ON / OFF button, the unit does not turn on.	1. Make sure you have inserted the batteries correctly. 2. Replace the batteries. Maybe they are exhausted.

XI. CLEANING AND DISINFECTION OF THE DEVICE

1. Initial manipulation: The maximum time after each use and cleaning the product before the next use is not more than 1 hour.

2. Before the cleaning: Remove the movable components – the silicone nozzle and the transparent cover. Take out the disposable cup.

3. Cleaning: Clean the aspirator’s main body with proper detergent (e.g. alcohol). Do not use corrosive chemicals to clean the device. Firstly, use a cotton swab or gauze moistened with 75% alcohol to remove any mucus from the silicone nuzzle and the transparent cover. Then rinse them under running water for about 3 minutes. Wipe the base for the disposable cup 2 times with gauze or cotton swab moistened with 75% alcohol. Leave them to dry or wipe them dry using clean dry cloth.

4. Disinfection: Sterilize the silicone nuzzle and the transparent cover in boiling water (100°C) or in steam sterilizer. Do not sterilize them in a microwave. The time for disinfection should not be more than 15 minutes in order to avoid damaging the parts. Use clean thongs to take the silicone nuzzle and the cover. Then wipe them dry with a gauze or cotton swab. Place them in the storage box to dry completely.

5. Maintenance before storage: Make sure that all parts are clean and dry and then assemble the aspirator as described in chapter VIII from this manual. And put it away in the storage box.

6. Storage requirements: Do not store the aspirator under direct sun light or in damp area.

7. Storage conditions:

-Room temperature: -20°C ~ 55°C (-4°F ~ 131°F)

-Relative humidity: ≤ 95%

-Pressure altitude: 700 hPa ~ 1060 hPa

XII. CONTENTS IN THE PACKAGE

- 1.Nasal aspirator – 1 pc.
- 2.Silicone probe – 3 pcs with different sizes – pink (0-3 months); blue (3 – 12 months); orange (12 months+).
- 3.Secretion cups – 25 pcs disposable.
- 4.2 pcs silicone sealing rings.
- 5.2 pcs AA alkaline batteries 1.5V.
- 6.User manual– 1 pc.

XIII. CLASSIFICATION

- 1.Internally powered equipment;
- 2.Type BF applied part;
- 3.IP22 ;
- 4.Disinfection: with ethyl spirit, boiling water or steam;
- 5.Category AP / APG equipment: N/A;
- 6.Intermittent operation

XIV.SYMBOL EXPLANATION

Symbol	Explanation
	Trade mark
	IEC 60417-5333, type attached part BF
	IEC 60417-5031, Direct current
IP22	Degree of protection against harmful ingress of water and particulate matter 2 Protected and against solid foreign objects of 12.5mm Φ and greater 2 Protection against vertically falling water drops when ENCLOSURE tilted up to 15°
	Refer to instruction manual
	DISPOSING: Do not throw this product with household waste. Collect separately, because special disposal of those kind of waste is required.
	Specific serial number
	Lot number
	Importer
	Model (Item) number
	Medical device
	Manufacturer
	Country of manufacture

Symbol	Explanation
	Symbol of European Authorized Representative, expressly indicate the name and address of European Authorized Representative after the symbol.
	The CE labelling certifies that the product complies with the essential requirements of EU regulations on medical device.
	Date of Manufacture
	Unique device identifier
	Indicates the need for the user to consult the instructions for use for crucial cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

XV. DECLARATION OF CONFORMITY

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare and clinic environments.

Warning: Don't hear active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EN disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment NCOO1, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

1.All necessary instructions for maintaining BASIC SAFETY and ESSENTIAL PERFORMANCE with regard to electromagnetic disturbances for the expected service life. Portable and mobile RF communication equipment may affect the performance of the NC001 avoiding strong electromagnetic interference when used, such as near mobile phones, microwave ovens, etc.

2.Guidance and manufacturer's declaration – electromagnetic emissions and Immunity.

Guidance and manufacturers' declaration – electromagnetic emissions and immunity

Table 1

Guidance and manufacturer's declaration – electromagnetic emissions	
Emissions test	Compliance
RF emissions CISPR 11	Croup 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not application
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not application

Table 2

Guidance and manufacturer's declaration – electromagnetic immunity		
Immunity test	IEC 60601-1-2 level test	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrical fast transient/ burst IEC 61000-4-4	Not application	Not application
Surge IEC 61000-4-5	Not application	Not application
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not application	Not application
Power frequency magnetic field IEC 61000-4-8	30 A/m 50Hz/60Hz	30 A/m 50Hz/60Hz
Conducted RF IEC61000-4-6	Not application	Not application
Radiated RF IEC61000-4-3	10 V/m 80 MHz - 2.7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz - 2.7 GHz 80 % AM at 1 kHz
NOTE: U_T is the a.c. mains voltage prior to application of the test level.		

Table 3

Guidance and manufacturer's declaration – electromagnetic immunity							
Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test frequency (MHz)	Band (MHz)	Service	Modulation		Distance (m)	Immunity test level (V/m)
	385	380-390	TETRA 400	Pulse modulation 18 Hz	1.8	0.3	27
	450	430-470	GMRS 460, FRS 460	FM ± 5kHz deviation 1 kHz sine	2	0.3	28
	710	704-787	LTE Band 13, 17	Pulse modulation 217Hz	0.2	0.3	9
	745						
	780						
	810	800-960	GSM 800/900, TETRA 800,iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18Hz	2	0.3	28
	870						
	930						
	1720	1700-1990	GSM 1800, CDMA 1900, GSM 1900, DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz	2	0.3	28
	1845						
	1970						
	2450	2400-2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217Hz	2	0.3	28

Guidance and manufacturer's declaration – electromagnetic immunity

Radiated RF IEC61000-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test frequency (MHz)	Band (MHz)	Service	Modulation		Distance (m)	Immunity test level (V/m)
	5240	5100-5800	WLAN 802.11 a/n	Pulse modulation 217Hz	0.2	0.3	9
	5500						
	5785						

XIII. DISPOSAL

It is required that you should not dispose of WEEE as unsorted municipal waste and collect such WEEE separately. In accordance with Directive 2012/19/EU and national disposal regulations regarding old electrical and electronic devices, be advised that such items must be disposed of in a special way within the European Union (EU). These regulations require the environmentally friendly recycling/disposal of old electrical and electronic devices. Such items must not be disposed of as domestic refuse. This has been expressed using the icon of the “crossed out trash can”.

The nasal aspirator (Model: NC001) contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it in household waste as unsorted municipal waste, but collect separately and take it to an appropriate local collection center. The nasal aspirator (Model: NC001) is classified as small equipment (5th category) according to ANNEX IV of Directive 2012/19/EU.

Before disposing of your nasal aspirator (Model: NC001), take away the batteries from it and dispose of the batteries and the nasal aspirator (Model: NC001) separately.

You can take away the batteries as follows – See Figure BI:

a. Open the lid on the back of the device.

b. Remove the batteries.

Batteries represent a hazard to the health and the environment! Never open, damage, or swallow batteries or allow them to pollute the environment. They may contain toxic, ecologically hazardous heavy metals. The removed batteries shall be disposed of at the point of sale or in the corresponding containers provided at collection points by local public waste authorities and treated according to 2006/66/EC.

The batteries and recyclable electronic waste shall be disposed of in a separate waste container. Packaging materials must be disposed of according to local regulations. Consult the authorized collection points for more information.

In Germany: For the recyclable electronic waste disposal, you can search the authorized collection points at the following website: <https://www.ear-system.de/ear-verzeichnis/sammel-und-ruecknahmestellen>. For batteries, you can search the authorized collection points at the following website: <https://www.ear-system.de/ear-verzeichnis/battghersteller#no-back>. Search method: using your postcode or the name of the city and state where you live to find collection points near you.

Other countries: For country-specific information on disposal, contact your local dealers or importer.

This appliance is manufactured by: Guangzhou Berrcom Medical Device Co., Ltd

Address: 38 Huanzhen Xi Road Dagang Town, Nansha, 511470 Guangzhou, Guangdong, PEOPLE'S REPUBLIC OF CHINA

The appliance is distributed in Bulgaria by: Moni Trade Ltd;

Address: Bulgaria, Sofia, Trebich, Dolo str. 1 **Tel.:** +359 2/ 936 07 90

Authorized representative of the manufacturer for Europe: Shanghai International Holding Corp. GmbH (Europe)

Address: Eiffestrasse 80, 20537 Hamburg, Germany

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Acest aparat este fabricat de: Guangzhou Berrcom Medical Device Co., Ltd

Adresa: 38 Huanzhen Xi Road Dagang Town, Nansha, 511470 Guangzhou, Guangdong, PEOPLE'S
REPUBLIC OF CHINA

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The Nasal Aspirator is a baby care product, which comes with a lot of advantages, such as its high efficiency and easy operation. Since babies up to one year old can only breathe through their nose, it is of great importance to clear their nasal discharge and mucus regularly when they are sick.

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Uređaj je proizveden od: Guangzhou Berrcom Medical Device Co., Ltd
Ovlašćeni predstavnik proizvođača za Evropu: Shanghai International Holding Corp. GmbH (Europe)